

geemarc



CLA 7


- Ⓜ User guide p.2
- Ⓧ Notice d'utilisation p.5

N e c k L o o p

1 - GENERAL

The mobile CLA 7 inductive loopset is for people with hearing aids. When you use the CLA 7, you can also speak directly to your caller by the microphone inside the system. So, the device works as a wireless system. Compatible with our products: Ampli DECT, CL200, 400, 500 & with most cellular phones (you may need to purchase a connector if your mobile does not have a suitable headset socket).

Don't forget to switch your hearing aid to T mode.

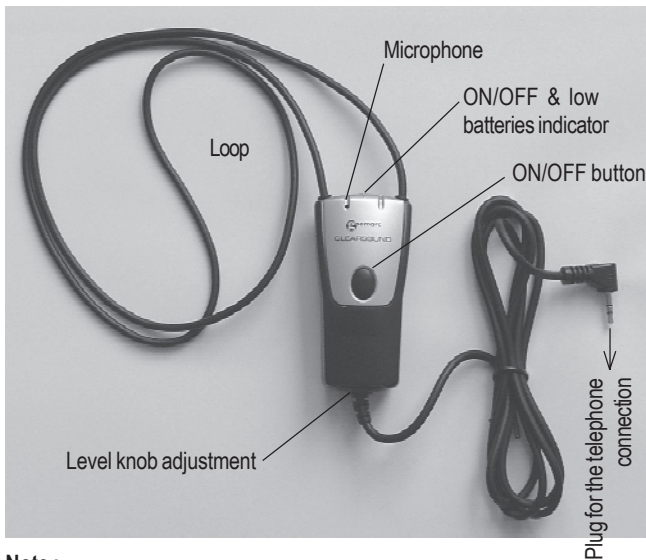

Note :

OFF : LED is off

ON : LED is green

Low battery : LED is red

2 - CONNECTING

- Open the battery compartment.
- Insert 2 AAA 1.5V type alkaline batteries without mercury, making sure the polarity is according to the diagram in the battery compartment.
- Replace the battery cover.

Now, the CLA 7 is ready to use.

Note :

- Always disconnect the plug of the CLA7 from the telephone before installing the batteries.
- Do not dispose of batteries in household rubbish. Pay attention to the battery regulations regarding battery disposal.

3 - MAKING AND RECEIVING CALLS

- Place the loop around your neck.
 - Connect the plug to the headset socket of your telephone(*).
- (see the instruction manual of your telephone - if you have a problem locating the socket).
- Switch your hearing aid to T mode.
 - Now you are ready to speak through your CLA 7. Press the **ON/OFF** button.

The green light will turn on.

- During the call, you can adjust the volume of the device by using the volume control knob located on the bottom of the CLA 7.
- On completion of the call, press the **ON/OFF** button.

The green light turn off.

Do not forget to turn OFF your CLA 7 at the end of the call. Failure to do this will drain the batteries after about 30 hours. Please consult your dealer for details of our high capacity rechargeable batteries and rapid battery charger. In the unlikely event that your dealer does not stock this item, please visit our website www.geemarc.com or call our HOTLINE

01707 38 44 38

4 - GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM

Please note: The guarantee applies to the United Kingdom only.

DECLARATION : Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC.



For product support and help visit our website at

www.geemarc.com

telephone 01707 384438

or fax 01707 372529