HUMANTECHNIK

PL-100

D

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Seite 2

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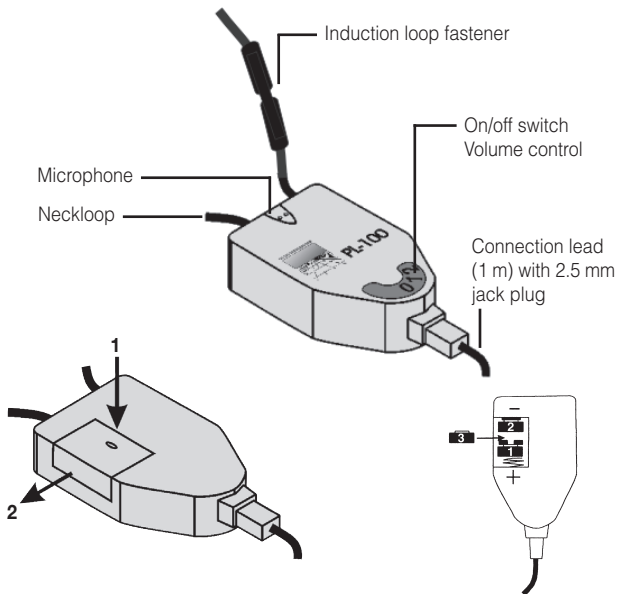
Inserting the batteries

The *PL-100* requires three hearing aid batteries (type 675, LR44)

1. Open the cover on the back of the *PL-100* by pushing it outwards
2. Insert the three batteries so the + sign matches up with the + mark in the battery compartment (spring contact end).
3. Insert the batteries in the following order:
 1. first battery between +ve terminal and plastic stops
 2. place the second battery against the -ve terminal
 3. insert the third battery between the first and second batteries.
4. Replace the battery cover.

Congratulations on purchasing the *PL-100*. We hope it will give you many years of reassuring operation.

Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *PL-100*

- Operating instructions

- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Functional principle

The *PL-100* generates a loop signal which can be received by a hearing aid with a built-in neckloop. This option is activated by setting the hearing aid to the »T« or »MT« position.

Making and receiving calls

1. Place the neckloop around your neck and plug the 2.5 mm jack plug into the connection jack socket provided on your mobile phone or its adaptor.
2. Switch on the *PL-100* and adjust to suitable volume.

3. Use your mobile phone in the normal way (making and receiving calls).

4. Switch off the amplifier by turning the volume control anticlockwise until you hear a click.

Note

Set the volume control of your mobile phone to the maximum so you can hear the voice of the person you are talking to clearly over any background noise.

Induction loop fastener

The induction loop fastener makes it easy to put on the device and minimizes the risk of accidents.

Maintenance and care

The *PL-100* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the *PL-100* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The *PL-100* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/ till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electro-

nic units. You protect the environment and health of your fellow men by the correct disposal of these products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.


Technical Data

| | |
|--------------------------|--------------------------|
| Height: | 53 mm |
| Width: | 48 mm |
| Depth: | 26 mm |
| Colour: | black |
| Weight: | 41 g |
| Inductive amplification: | approx. + 35 dB +/- 10 % |

This product complies with European Union norms. You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

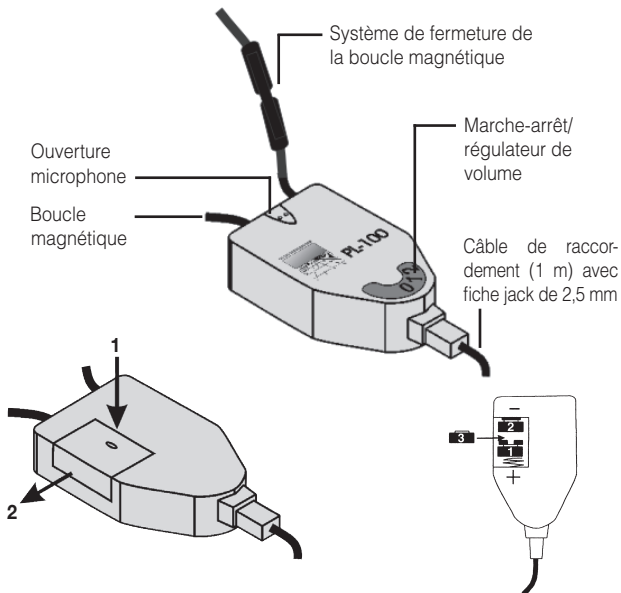Technical specifications subject to change without notice.

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Humantechnik GHL AG