

Telephone ring amplifier  
Amplificateur de sonnerie téléphonique  
Amplificador del timbre del teléfono



English Page 1


Français Page 13

Español Página 25

# DESCRIPTION


## Description

---

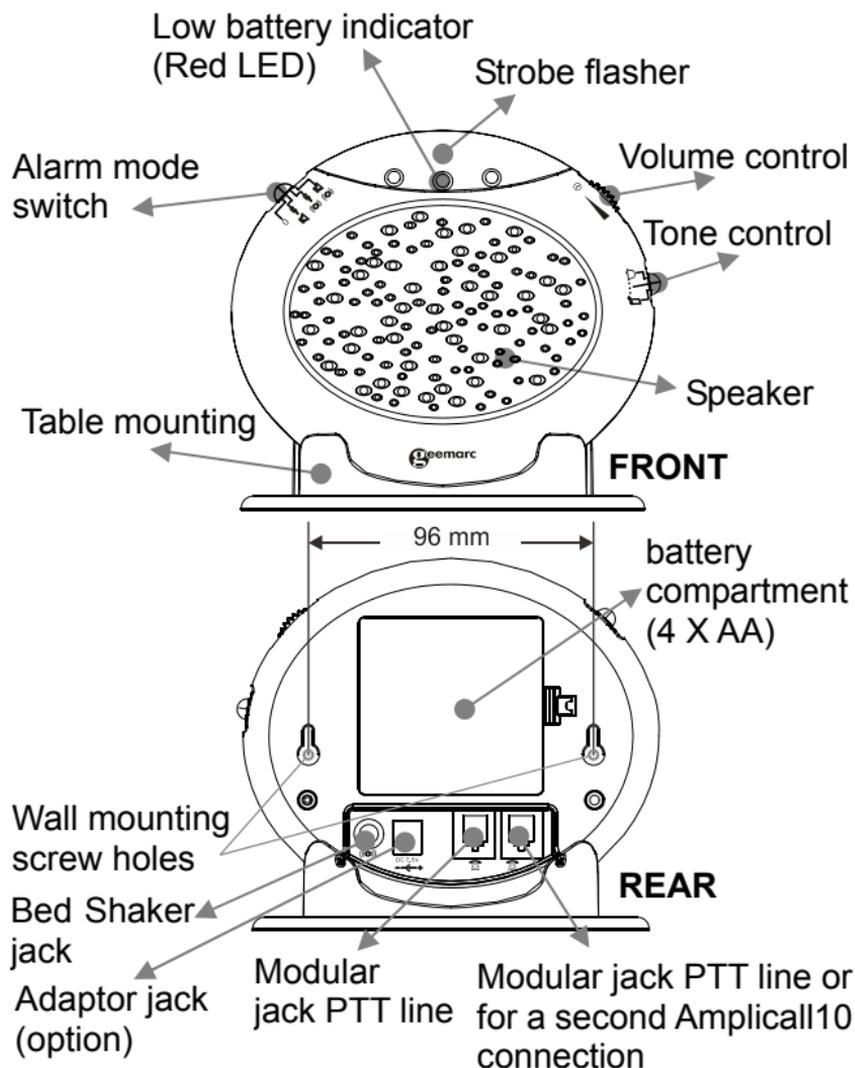

# INSTALLATION

## Installation (See drawing)

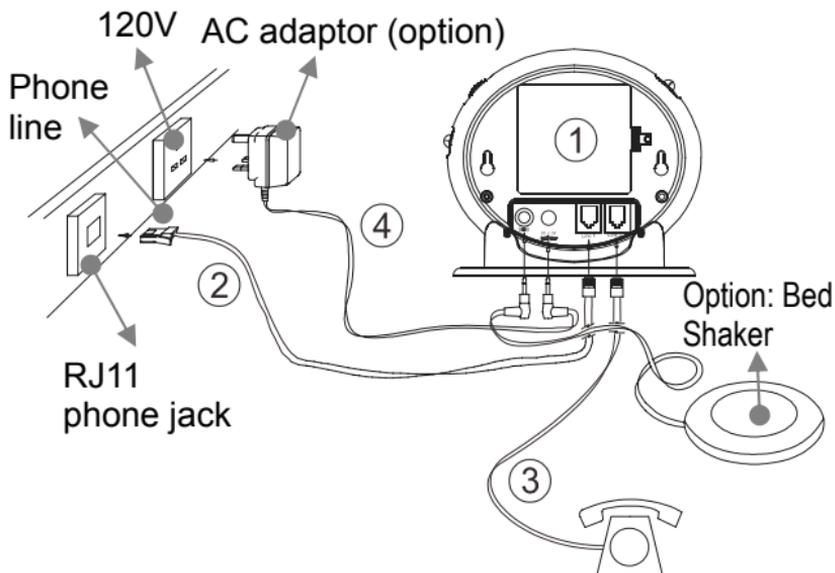

- 1) Insert 4 AA batteries into the battery compartment, for battery backup feature. Test batteries every year.
- 2) Connect the Phone cable to the line jack found at the rear of the Amplicall 10. Connect the other end of the cable to the on the wall.
- 3) Connect the telephone line cord from your telephone into the jack found at the rear of the Amplicall 10.
- 4) Plug the power cable into the power adaptor jack found at the rear of the Amplicall 10. Plug the adaptor into the mains supply at the wall (optional)

# INSTALLATION

## Alarm Operation

---

### 1 - SETTING THE ALARM MODE

Choose which alarm mode you want by moving the alarm mode switch to one of the following positions:

0 : no alarm

⚡ : both strobe flasher and sound alarm activated



⚡ : both strobe flasher and shaker (if connected)

⊕ : activated

⊕ : both sound alarm and shaker (if connected)

⊕ : activated

### 2 - VOLUME CONTROL SLIDE SWITCH

Set the volume of the alarm sound or power on/off.

### 3 - TONE CONTROL SWITCH (3 POSITIONS)

Set the tone of the alarm sound.

# INSTALLATION

## Wall Mounting

---

1. Drill two holes horizontally, 96mm apart, and leave the screws protruding from the wall by 5mm.
2. Hang the base on to the two screws and pull the base down to lock it into place.

## Table Mounting

---

If you want to mount the unit on a table, simply place it in the base cradle, and place the base cradle on the table.

## Change The Battery

---

The unit requires 4×AA batteries

1. Open the battery compartment door.
2. Insert 4 x AA batteries into the battery compartment.
3. Snap the battery compartment door back into place.

**Note:** when the red LED indicator flashes, it is time to replace the batteries.

When replacing the batteries, disconnect all telephone line cords from the modular wall jacks.

## Second Amplicall 10

---

Another Amplicall 10 can be added to the system (call 1-888-864-2446). Add the second piece on one second PTT socket.

# INSTALLATION

## Trouble shooting

---

Symptom	Solution
No incoming call yet the Amplicall10 rings automatically or rings continuously; the shaker vibrates automatically or continuously; the strobe LED flashes automatically.	<ol style="list-style-type: none"><li>1) Check the batteries are inserted correctly.</li><li>2) Check the power adaptor is connected correctly.</li><li>3) Turn off the power switch for 10 minutes and then turn it on.</li></ol>
The red low battery LED flashes automatically.	<ol style="list-style-type: none"><li>1) Replace the batteries.</li><li>2) Turn off the power switch for 10 minutes and then turn it on.</li></ol>
When there is an incoming call but no ringing can be heard.	<ol style="list-style-type: none"><li>1) Check it is connected to the power adaptor correctly.</li><li>2) Check the batteries are installed correctly.</li><li>3) Check if the volume switch is switched on.</li><li>4) Check it is connected to the line cord correctly.</li></ol>

# INSTALLATION

<p>When there is an incoming call but the shaker doesn't vibrate.</p>	<ol style="list-style-type: none"><li>1) Check it is connected to the shaker correctly.</li><li>2) Check the shaker socket to see if it has been damaged.</li><li>3) Check the alarm mode selection switch position is correct.</li></ol>
---	---